# How to use INDERMIL<sup>®</sup> flexifuze<sup>™</sup> in 3 easy steps.

### Step 1

Hold the applicator in one hand with the tip pointed upward and away from user and the patient to avoid inadvertent contact.



#### Step 2

Approximate the wound edges together. Press the device between forefinger and thumb on the dots to break the inner glass ampoule.



## Step 3

**Gently** squeeze the tube between thumb and forefinger unitl the adhesive appears at the brush tip. Apply the adhesive in a thin film over the length of the wound. Hold the wound closed for 60 seconds to complete the bond.

One device provides wound coverage of over 20cm.

Connexicon Medical Ltd., Synergy Building, Tallaght I.T., Dublin 24, Republic of Ireland Tel: +353 (1) 533 4995 www.connexiconmedical.com

 $\label{eq:INDERMIL*} INDERMIL* is a registered trademark of Connexicon Medical Ltd. flexifuze^{\texttt{m}} is a trademark of Connexicon Medical Ltd.$ 

© 2018 Connexicon Medical. Made in Ireland. [FF-HTUS-DIS-Rev005-180605] connexicon medical