



PATIENT INFORMATION GUIDE

INDERMIL® flexifuze™** topical tissue adhesive



If you have this card it's because flexifuze™ topical tissue adhesive has been used to close your wound.

The information contained within this leaflet will help you understand this closure method.

flexifuze™ is applied to the skin as a liquid which immediately begins to harden on contact with the surface of the skin.

When fully dry, the adhesive will appear wrinkled.


This is perfectly normal.

* Property of Connexicon Medical Ltd.

INDERMIL®
flexifuze™

Medtronic


CARING FOR WOUNDS

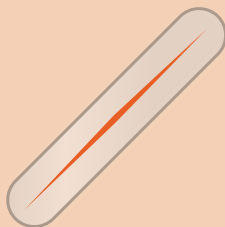

- Keep the wound as dry as possible until the adhesive barrier sloughs off in 5–8 days allowing for only occasional contact with water.



- You may immediately shower and bathe the wound site gently.

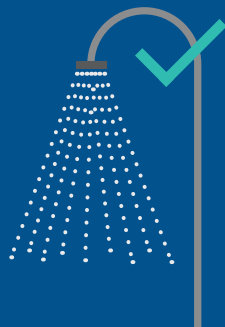

- Avoid scrubbing the wound site directly or excessive soaking in water (including swimming) until the adhesive barrier sloughs off and the wound site heals.




- Do not pull or pick at the wound or bandage.



- Do not apply any medications or cream to the wound.



- Keep the wound dry with a non-stick, non-medicinal and water resistant bandage, per your doctor's instructions.

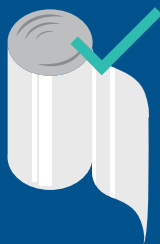

- Report any discomfort or other concerns regarding your wound to your doctor.



Medtronic